
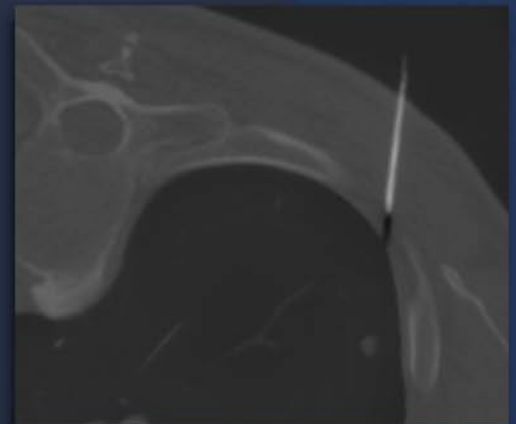
Operators Manual Bedienungshandbuch Manuel d'utilisation

Interactive Breath-Hold Control System (IBC)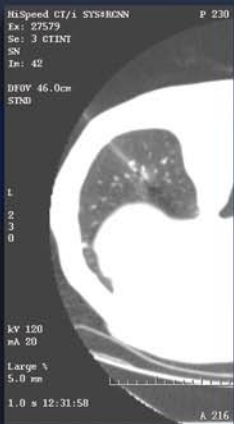

Product:
INTERACTIVE BREATH-HOLD CONTROL SYSTEM (IBC)
Operators Manual: Number 002 - Rev. 1.7

Manufacturer:
Mayo Clinic Medical Devices

English3

Deutsch.....19

Français.....35



ENGLISH

DEUTSCH

FRANÇAIS

Operators Manual

Interactive Breath-Hold Control System (IBC)

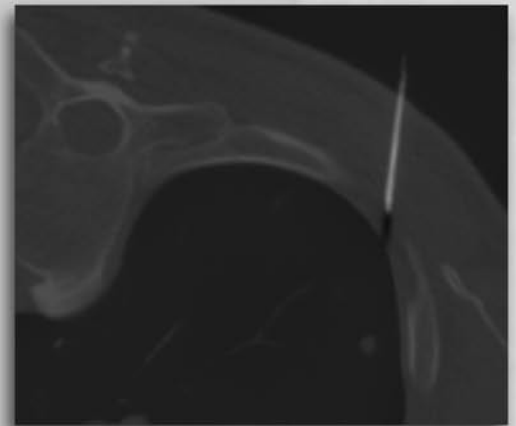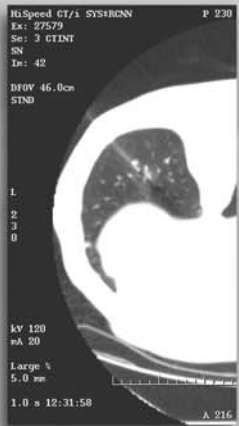Product:

INTERACTIVE BREATH-HOLD CONTROL SYSTEM (IBC)
Operators Manual: Number 002 - Rev. 1.7

Manufacturer:

Mayo Clinic Medical Devices

TABLE OF CONTENTS

Section 1 - Introduction	6
1-1 Operators Guide Information	7
1-2 Use of Symbols	7
1-3 UL 60601-1 and IEC 60601-1 Classification	8
1-4 Quality Assurance	8
1-5 Safety	8
■ 1-5-1 System Operating Safety	8
■ 1-5-2 Individual Safety.....	8
■ 1-5-3 Emergency Procedures	8
■ 1-5-4 Indications	9
■ 1-5-5 Equipment Safety.....	9
■ 1-5-6 Electrical and Mechanical Safety.....	9
■ 1-5-7 Modifications.....	9
■ 1-5-8 Reporting of Incidents	9
1-6 Shipping Contents	10
1-7 System Description	10
1-8 Environmental Requirements	10
■ 1-8-1 Operating Temperature Range	10
1-9 System Installation and Service	10
■ 1-9-1 Mounting Brackets	10
■ 1-9-2 Power Supply.....	11
■ 1-9-3 EMF or Other Interference	11
Section 2 - Setup and Instructions for Use	12
2-1 Introductory Instructions for Setup and System Use	12
■ 2-1-1 Switching On the Base Unit Power.....	12
■ 2-1-2 Switching On the Remote Display Unit Power	12
■ 2-1-3 Wireless Communication Pairing to Base Unit.....	12
■ 2-1-4 Operational Use	12
■ 2-1-5 System Status Indicators.....	13
■ 2-1-6 Wireless Lock LED.....	13
■ 2-1-7 Low Battery Indicator LED	13
■ 2-1-8 Battery Charging for Base Unit	13
■ 2-1-9 Battery Charging for Remote Display Unit.....	13
2-2 Positioning the System	14
■ 2-2-1 Prior to Using the System on an Individual.....	14
2-3 Positioning the Individual	14
■ 2-3-1 When the Individual Arrives.....	14

2-4 Positioning the IBC and Usage.....14

- 2-4-1 Positioning of the Bellows Belt Portion of the System14
- 2-4-2 Key Points14
- 2-4-3 Practicing With the IBC System.....15

2-5 Session Completion15

- 2-5-1 After the Session Is Complete.....15

2-6 General Troubleshooting15

- 2-6-1 How Do I Begin?.....15
- 2-6-2 How Can I Tell If the Remote Displays Are Charging Properly?.... 15
- 2-6-3 What If the Unit Will Not Turn On?15
- 2-6-4 Where Should the Belt Be Positioned On the Individual?16
- 2-6-5 What Does It Mean When the Lights Display Only
a Sequence of On/Off?16
- 2-6-6 What If the Unit Will Not Turn On?16
- 2-6-7 What Does It Mean When the Two Center Red Lights Blink?16

Section 3 - Maintenance.....17

3-1 System Care.....17

- 3-1-1 Cleaning Instructions17
- 3-1-2 Storage17

NOTE:
REBOOT SYSTEM BEFORE EVERY USE BY TURNING THE DOCKING STATION POWER OFF. WHILE IN THE OFF POSITION THE POWER CORD MUST BE REMOVED FROM THE DOCKING STATION AND RE-PLUGGED IN. POWER MAY BE TURNED BACK ON ONCE THIS ACTION IS COMPLETE.

Section 1 - Introduction

This manual describes the safety precautions, features, use, and care of the Mayo Clinic Interactive Breath-Hold Control (IBC) System. Please review this manual thoroughly before using the system.

If you have any questions or comments on this manual, or need any assistance with the use of the product, please contact a Mayo Clinic Medical Devices service representative at 866-677-8813 or visit the Web site at <http://www.mayoclinicmedicaldevices.com>.

1-1 Operators Guide Information

The operators guide provides information on how to use the Mayo Clinic Interactive Breath-Hold Control System. The user should read this guide before using the system.

1-2 Use of Symbols

Operators should understand the following symbols used for caution and warning explanations:

Battery

Bellows

ON

On

OFF

Off

Wireless

A hazard exists that will or could cause a minor injury

A hazard exists that could cause serious injury

Attention: Consult accompanying documents

Type BF applied part

Class II equipment

WEEE (Waste Electrical and Electronic Equipment)

Direct current

1-3 UL 60601-1 and IEC 60601-1 Classification

Class II equipment and internally powered equipment

Type BF applied parts

Suitable for continuous operation

This product conforms to the requirements of UL 60601-1 and IEC 60601-1 standards.

Authorized European Representative:

CEpartner4U
Esdoornlaan 13, 3951 DB Maarn
The Netherlands
Tel.: +31 (0)6-516.536.26

1-4 Quality Assurance

Prior to using the IBC System on an individual, power up the system and plug the bellows into the base unit. Expand and contract the bellows to verify that all remote displays are working properly. Place the bellows belt on the individual to ensure the readouts on all remote displays are synchronized.

1-5 Safety

This manual contains detailed information on the setup, positioning, and use of the Mayo Clinic Interactive Breath-Hold Control System. The instructions should be read carefully and thoroughly before using this system.

1-5-1 System Operating Safety

Individual safety and comfort should be the primary concern during use. Always follow proper safety, operating, and maintenance procedures to prevent exposure to electrical or mechanical hazards that may cause injury.

1-5-2 Individual Safety

Ensure that the individual is comfortably positioned.

NOTE:

This product contains natural rubber latex in the Bellows belt which may cause an allergic reaction.

1-5-3 Emergency Procedures

In the unlikely event that the system creates smoke, sparks, or makes an unusually loud noise, or if the individual requires emergency assistance:

- Stop the procedure if one is in progress.
- Remove the individual from the scan room if medical treatment is needed.

1-5-4 Indications

The Mayo Clinic Interactive Breath-Hold Control System will be used to help individuals control their breathing and achieve consistent breath-hold motion.

1-5-5 Equipment Safety

All personnel using the system should be instructed in the proper connection and handling of the system.

- Personnel should observe all warnings and cautions that appear in the guide.
- The system is not suitable for use in the presence of flammable anesthetics in combination with air, oxygen, or nitrous oxide.
- This equipment is not protected against ingress of water.

NOTE:

This equipment is rated for continuous operation.

NOTE:

For the disposal of this system at the end of its useful life, please contact a Mayo Clinic Medical Devices service representative at 866-677-8813.

1-5-6 Electrical and Mechanical Safety

- The system consists of both electrical and mechanical components. Service personnel must have specialized training to ensure the safe operating condition of the system. Therefore, only properly trained and qualified personnel should be authorized to service the system.
- The electrical and mechanical assemblies and parts of the system must be used with care and should be routinely inspected.
- Before using the system, visually check it for any external damage. Do not use the system if the housing or cable is broken.

1-5-7 Modifications

Any changes or modifications to the system must be approved by Mayo Clinic Medical Devices prior to installation.

1-5-8 Reporting of Incidents

- Users should contact their local field representative immediately to report an incident and/or injury to an individual, operator, or maintenance employee that occurred as a result of system operation.
- If an accident occurs as a result of system operation, do not operate the equipment until an authorized investigation is conducted.

1-6 Shipping Contents

Description	Quantity
Base unit/docking station	1
Bellows belt	2
Remote displays	3
Pad set	1
Operators manual	1
Table-mounting base	1
Mounting hardware for remote displays	5
Education video	1

1-7 System Description

The Mayo Clinic Interactive Breath-Hold Control (IBC) System is a system that will help individuals in whom breathing and breath-hold motion is a problem. The system consists of a strap that wraps around an individual's chest and is connected to a base unit and series of displays. The display indicates how deeply each breath is taken. While the Mayo Clinic Interactive Breath-Hold Control System may be used in conjunction with imaging systems, it is not connected to and does not control the imaging system in any way. The Interactive Breath-Hold Control System indicates the breathing level of an individual using a simple light display.

1-8 Environmental Requirements

Store the system in an air-conditioned scan room or equipment room.

1-8-1 Operating temperature range:

0 to 3,000 ft (914.4 m): 10 to 35 C (50 to 95 F)

3,000 to 7,000 ft (2,134 m): 10 to 32 C (50 to 89.6 F)

Operating and non-operating humidity range: 10% to 80% RH, non-condensing

Shipping/storage/non-operating temperature range: 0 to 50 C (32 to 122 F)

Shipping/storage humidity range: 10% to 90% RH, non-condensing

1-9 System Installation and Service

System installation and configuration require the services of your trained service representative. Please refer to the appropriate sections for system installation and configuration information.

1-9-1 Mounting Brackets

The mounting brackets for the remote displays must be installed by a technical representative of the user. The attachment must be made to provide a permanent anchor for the remote display. Note that the remote

display mates with the mounting bracket in a manner that locks the remote display into place on the mounting bracket. Insert the bottom of the mounting bracket first, pressing down against the spring-loaded edge. Rotate the remote display forward to allow the spring assembly to lift the bracket and hold it in place. Removal is accomplished by reversing the above steps.

1-9-2 Power Supply

The power supply for the base unit must be plugged into the AC wall power source when charging.

1-9-3 EMF or Other Interference

The base unit and remote displays are not affected by the electrical signals routinely found in any environment. The IBC System's wireless emissions are within the unlicensed 2.4GHz band.

This equipment has been tested and found to comply with the limits for medical devices to IEC 60601-1-2, Edition 2.0. This testing shows the device provides reasonable protection against harmful interference in a typical medical installation. However, there is no guarantee that interference will not occur in a particular situation. If this equipment does cause harmful interference to other devices or is negatively impacted by other devices, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the devices.
- Increase the separation between the devices.
- Connect the equipment to an outlet on a different electrical circuit.
- Consult the manufacturer for help.

Section 2 - Setup and Instructions for Use

2-1 Introductory Instructions for Setup and System Use

2-1-1 Switching On the Base Unit Power

When the base unit power is turned on, all LEDs on the base unit will be switched on for approximately two seconds. This action provides a check of the LEDs. After approximately two seconds, only the center LED will remain on if no errors are detected by the system. When an error is detected, alternating LEDs will be on, indicating an error. In this case, the docking station power should be turned off and the power cord unplugged from the docking station. Once these steps have been taken, plug in the power cord to the docking station and turn the power to the docking station back on. If the error persists, the base unit is unusable and must be replaced.

2-1-2 Switching On the Remote Display Unit Power

When the remote display unit power is turned on, all LEDs on the remote display unit will be on for approximately two seconds. This action provides a check of the LEDs. After approximately two seconds, only the center LED will remain on if no errors are detected by the system. When an error is detected, alternating LEDs will be on indicating an error. In this case, power should be turned off and turned on again. If the error code persists, the remote display unit is unusable and must be replaced.

2-1-3 Wireless Communication Pairing to Base Unit

Before use, remote display units must be “paired” with a base unit. Pairing provides the ability to have multiple base units within wireless proximity. Pairing associates a given remote display unit to a given base unit. Pairing is achieved by plugging a remote display unit into a base unit and turning power on to both the base unit and the remote display unit. Pairing is achieved when the remote display unit Wireless Lock LED is on.

2-1-4 Operational Use

IBC System operational use is achieved by pushing the reference button on either the base unit or a remote display unit. Pushing any reference button will open the solenoid and equalize the air pressure within the unit to identify a new reference level. Releasing a reference button will close the solenoid. All base unit LEDs will be on while the reference button on the base unit is pushed. All LEDs will be on for approximately one second when the reference button on a remote display unit is released. Following the release of the reference button, the center LED on both the base unit and the remote display unit will be on. Following this operation, the LEDs reflect the respiratory level indicated by the bellows indicating the new reference level at the display center.

2-1-5 System Status Indicators

The IBC has two system status indicators. These indicators are defined in 2.1.6 and 2.1.7 below.

2-1-6 Wireless Lock LED

The Wireless Lock LED has the same operation on both the base unit and the remote display unit. The Wireless Lock LED is on for approximately two seconds after the power switch is turned on. This action provides a check of the LED. The Wireless Lock LED on the base unit is on when the base unit is communicating. Likewise, the Wireless Lock LED on the remote display unit is on when the remote display unit is communicating.

2-1-7 Low Battery Indicator LED

The Low Battery Indicator LED has the same operation on both the base unit and the remote display unit. The Low Battery Indicator LED is on for approximately two seconds after the power switch is turned on. This action provides a check of the LED. The Low Battery Indicator LED blinks when the battery is charging. When the battery is extremely low the Low Battery Indicator LED will be blinking fast and the system should not be used at this time.

2-1-8 Battery Charging for Base Unit

The battery in the base unit will charge when the IBC power supply is connected to both the base unit and the wall outlet. When the AC connection is made, all LEDs will be on for approximately two seconds. After two seconds, only the center LED will be on. The base unit power switch may be in either the on or off position for charging to take place.

2-1-9 Battery Charging for Remote Display Unit

The remote display unit must be plugged into the base unit. The battery in the remote display unit will charge when the AC power supply is plugged into both the base unit and the wall outlet. When the AC connection is made, all LEDs will be on for approximately two seconds. After two seconds, only the center LED will be on. The remote display unit power switch may be in either the on or off position for charging to take place.

2-2 Positioning the System

2-2-1 Prior to using the system on an individual:

- Have individual view the education video.
- Assemble the Interactive Breath-Hold Control System:
 1. Place the bellows belt and base unit on the table.
 2. Place the bed mount display on the table, under the individual's pillow and table pad.
 3. Place remote displays where they can be viewed by any third party who may need to monitor or help the individual achieve consistent breathing and breath-holding.
 4. Turn on the power for the base unit and all remote displays.

2-3 Positioning the Individual

2-3-1 When the individual arrives:

- Place the bellows belt on the individual immediately upon his or her entering the monitoring suite so that the belt has time to acclimate to the individual's body temperature. The belt's position can be adjusted later if necessary to ensure proper monitoring.
- Press the reference button on any remote display or the base unit to begin the monitor function.
- Familiarize the individual with the Interactive Breath-Hold Control System and how the remote display changes as the individual breathes.

2-4 Positioning the IBC and Usage

2-4-1 Positioning of the bellows belt portion of the system:

- Ensure that the bellows belt is positioned on the individual's bare skin around the lower chest over the bottom of the ribs near the diaphragm.

2-4-2 Key Points:

- The bellows belt should be moderately stretched and secured tightly enough so that it does not move in position on the individual during monitoring.
- The bellows belt should be able to expand and contract as the individual breathes (watch for movement of the bellows as the individual breathes).
- Verify that the bellows belt is able to expand. Make sure the individual is not lying on any portion of the bellows belt or the connecting rubber tubing.
- Verify that the rubber tubing connected to the bellows belt is positioned off the floor and free of kinks or other obstructions to airflow.
- Explain to the individual that it is very important to lie still and to keep the bellows belt in place.

2-4-3 Practicing with the IBC System:

- Instruct the individual to view the remote display as he or she breathes. Green and red LEDs will light up as the bellows belt expands and contract with respiration.
- Communicate to the individual that small, comfortable inspirations in which the individual can hold his or her breath for an extended period of time are ideal. During the breath-hold, the goal is to keep the center red diode lit. Have the individual take a comfortable inspiration and hold it. Center the system and remote displays by pushing and releasing the reference button on the base unit or any remote display. This will establish the reference breath-hold position that the individual will return to. Have the individual practice this technique a few times.
- After the individual is comfortable with the system, tell him or her to continue returning to the breath-hold position in which the center diode is lit whenever he or she is instructed to take a breath and hold it.

2-5 Session Completion**2-5-1 After the session is complete:**

- When turning the base unit power switch off, all base unit LEDs will be off, except the center LED when the battery is charging. When turning the remote display unit power switch off, all remote display unit LEDs will be off, except the center LED when the battery is charging.
- Remove the bellows belt from the individual.
- Remember to clean and disinfect the bellows belt before returning it to its storage place.
- Remove the base unit and remote displays; recharge each if necessary prior to storage.

2-6 General Troubleshooting**2-6-1 How do I begin?**

Position and connect the IBC System as shown in the Operators Manual and video. Ensure that the bellows belt fits around the individual snugly, but comfortably. Turn on power to the base and remote display units; push the reference button on the base unit or any remote display.

2-6-2 How can I tell if the remote displays are charging properly?

When turned off and positioned properly within the base unit, an amber indicator light on the side of the remote display should be lit.

2-6-3 What if the unit will not turn on?

It is possible the batteries are not charged. Ensure all units are plugged in during battery charging.

2-6-4 Where should the belt be positioned on the individual?

Reliable breath control has been achieved with the belt positioned around the ribs in the lower chest at or near the level of the diaphragm. The exact position can be adjusted.

2-6-5 What does it mean when the lights display only a sequence of on/off?

The hardware is failing and must be replaced.

2-6-6 Why doesn't the reference button work with the base unit attached to AC power?

If you are recharging the batteries with the base unit main power in the off position, the system software considers the system not in use. With the base unit turned off, the reference button will have no effect on the system.

2-6-7 What does it mean when the two center red lights blink?

The remote display has temporarily lost the wireless connection. Reposition the display on base unit until the system reconnects.

NOTE:

REBOOT SYSTEM BEFORE EVERY USE BY TURNING THE DOCKING STATION POWER OFF. WHILE IN THE OFF POSITION THE POWER CORD MUST BE REMOVED FROM THE DOCKING STATION AND RE-PLUGGED IN. POWER MAY BE TURNED BACK ON ONCE THIS ACTION IS COMPLETE.

Section 3 - Maintenance

3-1 System Care

Detach the system from AC power before attempting to clean it. Do not reattach after cleaning until the system has dried completely. Having the system attached to the AC power during cleaning or when it is wet may result in electrical shock.

3-1-1 Cleaning Instructions

- The recommended approach to clean the latex rubber bellows belt is as follows:
 1. Wipe down with mild soap and water.
 2. Wipe with clean dry cloth.
 3. Follow up with wipe down of 70% isopropyl alcohol (IPA) and 30% water to expedite drying time.
 4. Allow five minutes to air-dry.

- The recommended approach to clean the IBC electronic enclosures is as follows:

Cleaning via a clean damp cloth (not immersion) using any of the following solutions and then allowing components to air dry:

- 10% bleach mixed in 90% tap water
- 30% isopropyl alcohol (IPA) in 70% tap water
- 3M-brand HB Quat Disinfectant Cleaner

NOTE:

DO NOT RUB DECALS, AS THEY WILL RUB OFF OVER TIME.

Under no circumstances should the system be placed into any type of sterilizer!

3-1-2 Storage

Store the system in an air-conditioned scan room or equipment room. (Refer to page 10, section 1.8 for room environmental requirements.)

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